

# Quick Patient Setup

- 1) Turn Wrist Pulse Oximeter on by pressing the white center button  
- hold for five seconds.
- 2) Screen will show oximeter parameters.
- 3) Scroll down to record by pressing white button.
- 4) Hard press (press and hold).
- 5) Short press white middle button Yes to record.
- 6) Press again to record (waiting, recording).
- 7) Place wrist unit on either right or left wrist.
- 8) Insert index finger into gray rubber sensor with cord on top.
- 9) You are now ready to record your night events.  
  
(Note - Screen will go blank but is still recording).

## Once Your Night Study is Complete - Turning Unit Off

- 10) Press the white center button and hold.
- 11) When you see the oximeter parameters scroll down to record.
- 12) Press record stop recording push yes.
- 13) Scroll down to Power off push power off.